Opalescence® Boost

Tooth Whitening System

Description:

Opalescence® Boost is a 38% hydrogen peroxide power bleaching gel. The distinctive syringe-to-syringe jet mixing ensures freshness for each application and exact dosing of the activator to provide the most effective product possible. One barrel contains a unique chemical activator with 1.1% fluoride and 3% potassium nitrate, which have been demonstrated to reduce caries susceptibility, lower sensitivity and improve enamel microhardness. The other barrel contains concentrated hydrogen peroxide. When mixed, Opalescence Boost is ready for use. Opalescence Boost is gluten-free.

OpalDam® is a unique, light-reflective, passively adhesive (sealing) light cured resin. Its methacrylate base is designed with sufficient strength to maintain a barrier yet is easily removed from embrasures and undercuts.

Indications:

Opalescence Boost power bleach is for in-office use only. It is used for dentist-applied bleaching of one or more teeth, parts of a tooth, and/or accelerated chairside whitening technique. Opalescence Boost is also used on non-vital teeth, including intracoronal bleaching in office.

Opalescence Boost is an alternative, conservative method for treating dark, discolored teeth (compared to crowns, veneers, etc.). This includes discolorations cause by congenital, systemic, metabolic, pharmacological, traumatic or iatrogenic factors such as dental fluorosis, jaundice, tetracycline and adult minocycline stains, porphyria, trauma and erythroblastosis fetalis.

Because restorative materials will not whiten, we recommend whitening teeth before esthetic restorative placement (wait two weeks after bleaching procedures before placing adhesive restorations). Bleaching problematic teeth to a more natural shade will optimize shade matching. Use Opalescence Boost alone or in conjunction with the dentist-supervised Opalescence Take-Home Bleaching System. Our experience shows that most treated teeth continue to whiten 12-24 hours after treatment. OpalDam is used to protect soft tissue adjacent to the teeth during in-office bleaching, microabrasion, etc.

Preliminary Procedures:

1. Remove any calculus or extrinsic stains before the bleaching appointment. This is ideally done prior to the bleaching appointment to prevent sulcular seepage at time of OpalDam application and bleaching treatment.
2. Determine and record shade before bleaching.
3. **WARNING: Shield patient's and clinician's eyes with protective eyewear with side shields during mixing and applying.**
4. Place Ultradent IsoBlock (Fig. 3).

Isolation:

Isolate teeth to be bleached with OpalDam light cured resin (Fig. 1) or with conventional rubber dam (we recommend powder-free DermaDam®) used with OraSeal® Caulking for maximum rubber dam seal. If bleaching only one or a few teeth, OpalDam is the gingival barrier of choice.

1. Isolation Procedure Using OpalDam:

- a. Remove luer lock cap and attach a disposable Micro 20 gauge tip securely onto OpalDam syringe.
- b. Place self-supporting plastic cheek retractors. Completely rinse and air dry teeth, paying special attention to the gingival third.

Note: When isolating the mandibular arch, prevent saliva from flowing through embrasures of anterior teeth by using saliva ejector and cotton rolls in the sublingual region. Express the OpalDam through the embrasures, onto cingulum, and on adjacent gingiva. Light cure.

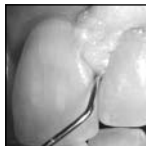
Fig. 1 Express OpalDam resin to build a 4-6mm by 1.5-2mm thick strip onto gingiva. Lap approximately 0.5mm onto enamel.

- c. Express OpalDam resin to build a 4-6mm by 1.5-2mm thick strip onto gingiva. Lap approximately 0.5mm onto enamel. If excess resin flows onto enamel, "tack" area for 5 seconds with curing light and use a hand instrument to trim back excess. Extend resin beyond the last tooth to be bleached. Where open embrasure(s) exist, express resin through the opening onto lingual, filling embrasure space completely. This will protect interdental papilla and lingual gingival tissue. Resin buttons extending through gingival embrasures add retention to OpalDam barrier.
- d. After resin barrier is applied, cure resin with a quality curing light 20 seconds (Fig. 2).

Fig. 2 Cure resin with a quality curing light 20 seconds.

- Clean teeth with prophyl paste to remove any excess resin film. Use caution not to dislodge cured OpalDam barrier.
- e. OpalDam works nicely for full arch bleaching. Place a tongue/bite block (we recommend IsoBlock - Fig. 3.) in molar region and instruct patient to hold it in place with gentle but firm biting pressure. Place cotton rolls into vestibule.

2. Alternative Isolation Procedure Using Rubber Dam:

- a. For safety, ligate rubber dam clamps with floss. Place appropriate clamps, bilaterally, one tooth posterior to teeth being whitened.
- b. Punch holes and position rubber dam over clamps and teeth.
- c. Before attaching rubber dam onto frame, extrude an even bead (~2-3mm wide) of OraSeal Caulking along contours of free gingiva, including interproximal areas, to develop a seal.
- d. Attach rubber dam to frame. Using a rubber dam instrument or an Ultradent® Slide Packer and air, invert rubber dam border around teeth for maximum tooth exposure and optimum rubber dam seal. Check rubber dam for tears or areas of possible leakage.

Fig. 3 Place Ultradent's IsoBlock.

Application of Opalescence Boost:

1. Allow contents of syringe to come to room temperature before mixing the two chemicals together. To activate, press chemical from the red syringe to the clear syringe with thumbs. Mix activator with bleaching agent by quickly and forcefully pressing clear plunger into middle clear barrel. This will rupture the internal plunger membrane which will allow mixing the activator and bleaching agent. Reverse action and mix a minimum of 25 times rapidly (12-13 each direction/side). Press mixed chemical into red syringe. Remove clear syringe and dispose. Twist Micro 20g FX tip onto red syringe. Mixed Opalescence Boost expires ten (10) days refrigerated, after activation.

Fig. 4 Activate by pressing plunger 12-13 times each direction/side. Finish with total contents in red syringe. Remove clear syringe and dispose. Twist tip firmly onto red syringe.

2. Check material flow on a cotton gauze or mixing pad to ensure evenness of flow before using intraorally. If resistance is felt, DO NOT proceed. Attach a new tip and test again before application.
3. Apply Opalescence Boost 0.5 to 1.0mm thick layer (Fig.5). Place the gel on the labial surfaces. Extend slightly onto incisal/occlusal edges.
4. For optimum effectiveness stir/agitate every 5 minutes.
5. Applications typically last between 10-20 minutes. Prior to rinsing, remove Opalescence Boost from teeth using surgical suction only (Fig. 6). To prevent Opalescence Boost from splattering onto soft tissues, DO NOT use water during initial suction. Ultradent recommends using a Luer Vacuum adapter with SurgiTip™. Reapply Opalescence Boost (Steps 3-5) three to four times, or until desired results are achieved. Do not exceed six applications of Opalescence Boost in one visit. After all visible gel is removed, follow with thorough rinse directing water towards incisal edge, and high-volume suction. Air dry teeth. Use caution to not dislodge resin barrier.
6. Evaluate the shade change after each application. The total procedure typically lasts approximately 60 minutes. If additional whitening is desired, and no significant tooth sensitivity is observed, schedule patient for repeat treatment at least 3 days following the treatment.

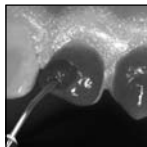
Fig. 5 Apply Opalescence Boost 0.5 to 1.0mm thick layer. Place the gel on the labial surfaces. Extend slightly onto incisal/occlusal edge.

Cleanup:

1. For OpalDam:

- After thoroughly vacuuming off whitening gel (Fig. 6), rinse teeth with air/water syringe and vacuum (Fig. 7).
Lift OpalDam barrier from surface using an explorer or instrument. All or large pieces will usually lift off at one time.
- Remove OpalDam interproximally using explorer and/or floss. Rinse interproximally with firm air/water spray.

2. For Rubber Dam:

- After thoroughly vacuuming off whitening gel, rinse teeth with air/water syringe and vacuum. Remove clamps and rubber dam.
- Remove bulk of OraSeal with a spatula. Remove residual by rinsing while using a soft toothbrush and vacuum.
A firm air/water rinse and dental floss may be necessary interproximally.

Notes:

- Opalescence Boost may be used in conjunction with the take-home, 10, 15, 20, and 35% carbamide peroxide Opalescence® Tooth Whitening Gel or Opalescence® Freshwhite™ Supreme to complete the whitening treatment. Usually 2-4 night's treatment is sufficient. Wait 24 hours after in-office procedure before starting patient on dentist-supervised home bleaching materials.
- Should patient experience post-op sensitivity, dentist can prescribe UltraEZ® potassium nitrate gel in a tray one hour per day until condition subsides.
- Opalescence Boost is neutral pH (7.0). Other power bleaching gels are usually acidic and require difficult, messy chairside mixing.
- Etching before bleaching does not potentiate the bleaching process and is not recommended with Opalescence Boost. Opalescence Boost is not dependent upon opening the "enamel pores" with acid.

Nonvital Teeth:

- Open lingual access to pulp chamber and remove all composite, base material, and gutta percha 2-3mm apical to the CEJ. Place a glass ionomer base 1mm thick to ensure a barrier between the "sealed" root canal and bleaching gel.
- Follow "Isolation" instructions above for OpalDam technique.
- Apply OpalDam resin to both the labial and lingual gingiva of the dry, nonvital tooth, extending about 0.5mm onto tooth. Light cure 20 seconds.
- Express Opalescence Boost into the opened pulp chamber and onto the labial surface. After 10 minutes, remove Opalescence Boost from teeth using suction only. Follow with a thorough rinse and high-volume suction. Air dry teeth. Use caution not to dislodge resin barrier.
Note: Use Opalescence® Endo for "walking" type treatment with non-vital teeth.
- Repeat steps 1-4 until desired results are achieved. If significant results are not achieved in 20-30 minutes (total time per tooth), discontinue.
- When desired color is reached, follow "Cleanup" procedure as previously instructed. Note: If pulp chamber is left open for continued tray-delivered bleaching, instruct patient to remove cotton pellet, express gel into the open chamber, then place a filled tray. Instruct patient to insert a new cotton pellet following tray bleaching procedure. Do not treat patient for restorative composite until two weeks after bleaching is completed.

Single- or Partial-Tooth Bleaching:

- The recommended barrier technique for single tooth bleaching is "Isolation Procedure Using OpalDam". The rubber dam technique may be used if desired.
- Apply Opalescence Boost to discolored or stained tooth (or part of tooth), according to directions above.
- Repeat procedure until desired results are achieved.
- "Clean up" as instructed in previous section.

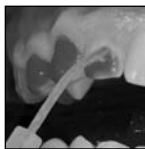
Fig. 6 Remove Opalescence Boost using suction with UltraDent's SurgiTip or using a surgical aspiration tip.

Fig. 7 Rinse thoroughly directing water toward high-volume suction.

Warnings/Precautions:

- WARNING: Clinician and patient must wear protective eyewear with side shields during mixing and applying!**
- Do not mix chemicals in syringe when cold. The product is thicker when cold and the force required to mix can break the syringe.
- Check material flow on an inert surface (i.e. mixing pad) to ensure evenness of flow before using intraorally. If resistance is felt, DO NOT proceed. Attach a new tip and test again before using on patient.
- After mixing, Opalescence Boost is good for 10 days refrigerated. After 10 days, discard using proper disposal methods as syringe may develop pressure. Materials should be diluted with water and disposed down a drain or toilet followed by a few liters/quarts of additional water. DO NOT discard to garbage cans containing burnable material such as paper, cardboard, rubber, leather, cotton and the like as spontaneous combustion can result. Additionally do not bring this strong peroxide into contact with such flammable materials during application as combustion may occur.
- Wear protective gloves and eye covering when handling this product! Hydrogen peroxide is a strong oxidizing agent and can cause burns to soft tissues (i.e. gingiva/lips). Keep isolated to area of treatment. Vacuum gel off teeth before rinsing with water to prevent splattering.
- Gingival and general oral health should be confirmed before treatment.
- Although there is no evidence to the contrary, because of safety precautions, DO NOT use Opalescence Boost on patients who are pregnant.
- Opalescence Boost is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.
- Restorations should be water-tight, and all exposed sensitive dentin should be covered. If areas are sensitive or potentially so, treat with proper restoration, dentin bonding agent, etc.
- Larger recurring bubbles in Opalescence Boost may indicate leakage of peroxide gel onto gingiva or presence of organic debris. Vacuum off gel, wash, dry, and inspect. If leakage appears to be the problem, use ViscoStat Clear on gingiva to eliminate sulcular fluid and to increase adhesion and seal of OpalDam resin. Once ViscoStat Clear has been thoroughly rinsed from area, apply OpalDam.
- Areas of hypocalcification, clinically not visible, occasionally exist. Because of mineralization differences, these areas will whiten faster than normal enamel and will become more visible during bleaching. Continue bleaching until remaining tooth surface more closely blends with hypocalcified area (additional blending usually occurs later during normal rehydration following completion of bleaching). Additional appointments and/or dentist-supervised tray bleaching may be required. Re-evaluate in two weeks when color has stabilized.
- Determine and understand patients' expectations before beginning treatment. Inform patients that existing restorations, crowns, and bridges do not lighten. Inform patients that because of unforeseen factors or enamel variations, some teeth may not meet desired expectations.
- Caution should be used to prevent transfer of Opalescence Boost gel to adjacent tissue(s) from inadvertent contact from gloved finger, vacuum tip(s), etc. Affected tissues should immediately be rinsed thoroughly clean with water.
- Clinician should wait two weeks following the bleaching procedure before matching restorations to tooth shade to ensure stabilization of color and to ensure all peroxide has leached from the enamel and/or dentin. Residual peroxide (oxygen) in the tooth will prevent adequate bonding.
- Monitor patients' bleaching progress during procedure and with take-home gel to prevent over-bleaching and/or excess enamel translucency.
- Some enamel stains do NOT respond to bleaching. Inform your patient of this possibility before treatment. Opalustre® microabrasion slurry may be used on surface stains that do not bleach.
- Opalescence Boost is supplied in preloaded syringes. Material is NOT to be injected. When empty, dispose of syringes properly. Avoid disposing of syringes in receptacles containing combustible products. Strong peroxides can initiate combustion.
- Do not use OpalDam on patients with known sensitivity to resins. If dermatitis, rash or any other allergic reaction develops, wash area with water and refer patient to a physician.
- Partially used syringes may be cleaned and disinfected between patients. Use Ultradent® Syringe Covers sealed with the Ultradent Impulse Sealer for a protective barrier against cross-contamination.
- Do not anesthetize teeth for in-office bleaching.

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© Copyright 2007 Ultradent Products, Inc. All Rights Reserved. All rights reserved.